
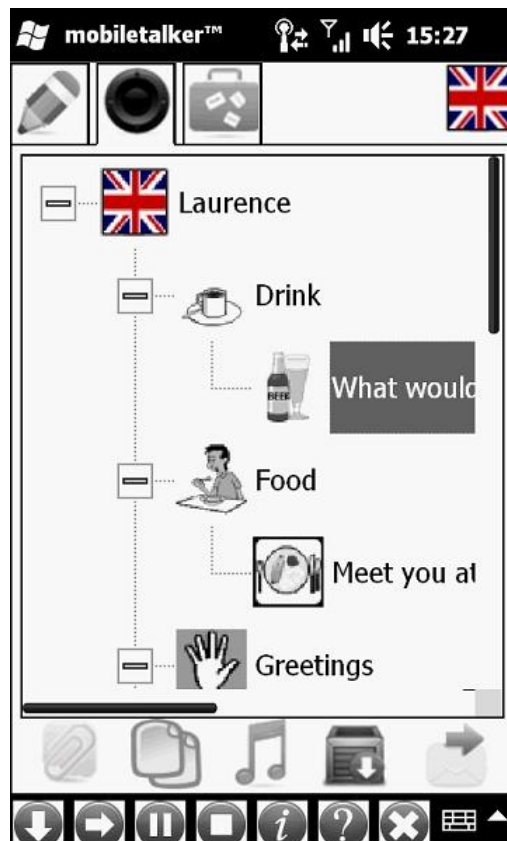
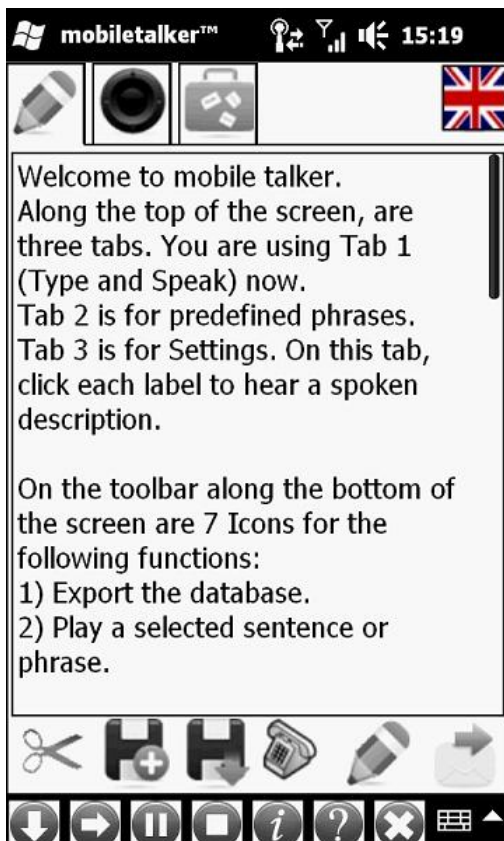
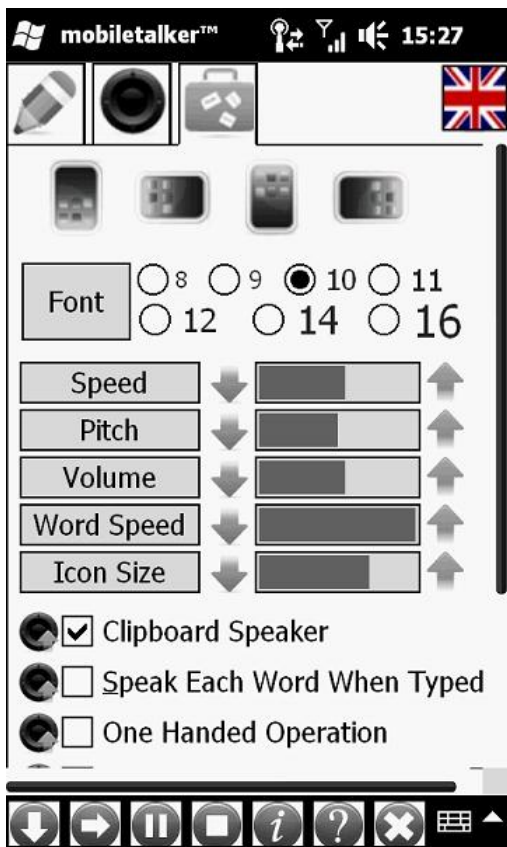


# MobileTalker Version 3








**Mobile Talker** is designed to enable a non-verbal (aphasic) user to speak anything they may wish to say, or the dyslexic user to hear text messages, and improve their use of written language. The user can type a word, phrase or sentence and speak it using Text-to-Speech or they may select pre-defined words, phrases and sentences from a categorized list. Optionally, the user can enable Clipboard Speaker mode, whereby any text copied to the clipboard from any other program running on the Windows Mobile device will be spoken.

|                               |  |
|-------------------------------|--|
| <b>Trial Period</b>           | Until a licence is purchased for <b>Mobile Talker</b> , you can run it 20 times for up to 1 hour each time. After this it will no longer work, even if you re-install it.  |
| <b>Installation</b>           | <b>Mobile Talker</b> is designed to run on Windows Mobile PDA's or Windows Mobile Phones running Windows Mobile versions 5, 6.1 or 6.5.<br><b>Mobile Talker</b> is distributed on an SD Memory card <b>or</b> as an a compressed zip file that can be extracted to an SD Memory card suitable for your Windows Mobile device.<br>Insert the SD Card into your device, and an autorun program will check and setup your device.<br>Follow the on-screen instructions. |
| <b>Registration</b>           | To register <b>Mobile Talker</b> , run <b>Mobile Talker Register</b> from the windows Start Menu, Programs. Before you run <b>Mobile Talker Register</b> , ensure that your Windows Mobile Device's <b>Owner Name</b> matches the <b>Licence Owner Name</b> that was agreed with your supplier when you purchased <b>Mobile Talker</b> .   |
| <b>Mobile Talker Register</b> | Ensure that your device is able to connect to the internet. When <b>Mobile Talker Register</b> starts, select your language, by tapping the country flag. e.g.  for English. Click the <b>Register Online</b> button. If registration fails, try a few more times before requesting support.  |
| <b>Support</b>                | For support please email your supplier, or <a href="mailto:support@bostock.com">support@bostock.com</a> with a subject of <b>Mobile Talker</b> .   |












## Toolbar

|  |   |
|--|---|
| <br><b>Export</b> | <p>Exports the current database of phrases to text files so they may be backed up or used on another installation. The main files that are created are: Languages.sql, Categories.sql, Phrases.sql. Depending on the Text-to-speech engine in use, other files may be created.</p> <p>These files enable the database to be recreated if it should become corrupt. The database is stored in the \My Documents folder and is called Mobile Talker.sdf. We recommend that you make regular backups of the database to your PC.</p> |
| <br><b>Play</b>   | <p>Play the current text. What is played, will depend on which tab is selected, and what text on the tab is selected.</p> <p>On Tab 1, Type &amp; Speak, if no text is selected, all text will be spoken. If only part of the text is selected, then it will be played.</p>   |
| <br><b>Pause</b>  | <p>Pauses the playing text. Pressing the Pause again (or the Play button) will continue playing the paused text.</p>  |
| <br><b>Stop</b>   | <p>Stops the playing text.</p>  |
| <br><b>About</b>  | <p>Display information about the <b>Mobile Talker</b> program, the licence, and the device.</p>   |
| <br><b>Help</b>   | <p>Displays this help file.</p>   |
| <br><b>Exit</b>   | <p>Exit the program.</p>  |






**Language**

The country flag will appear at the top right hand side of the screen for each installed voice. If more than one flag appears, you can change language by tapping the appropriate flag.

**Main Tabs****Type + Speak TAB**


|  |  |
|--|--|
| <b>Sentence Input Box</b>  | Anything typed into this text box can be spoken by pressing the  (Play) Button. Clear the entry box with the  (Clear) button |
| <br><b>Clear button</b>     | Press to clear the Sentence Input Box. If some of the text is selected, the clear button will only clear the selected text.  |
| <br><b>Load button</b>      | Load the Sentence Input Box from a text file.  |
| <br><b>Save button</b>      | Save the text in the Sentence Input Box to a text file.  |
| <br><b>Keyboard button</b> | Press to display or Hide the on-screen keyboard.   |
| <br><b>SMS button</b>     | Press to send all, or the selected text in the Sentence Input Box using SMS Text messaging.  |

**Phrases TAB**

|   |   |
|---|---|
| <br><b>New or Edit</b>   | Creates a <a href="#">new category</a> or a <a href="#">new phrase</a> , or if a phrase is already selected, it will be edited.                     |
| <br><b>Copy</b>          | Copy the selected category or phrase to the clipboard.  |
| <br><b>Pronunciation</b> | Displays the <a href="#">pronunciation window</a> . You need to know the specific syntax for your Text-to-speech engine, before using this feature. |
| <br><b>Delete</b>        | Delete the selected category or phrase.   |
| <br><b>SMS button</b>    | Press to load the <a href="#">Send SMS</a> window, so you can send the selected phrase using SMS Text messaging.                                    |



## Settings TAB

|   |  |
|---|--|
| <br><b>Orientation</b> | Choose your preferred orientation.   |
| <b>Font Size</b>  | Select your preferred font size. Note, that on some devices, using the larger fonts may mean that some text fields are truncated.  |
| <b>Speed</b>  | Set your preferred speaking speed.   |
| <b>Volume</b>   | Set your preferred volume. Note that this is independent of the devices volume. To achieve the maximum volume, set both this and the devices' volume control to the maximum.   |
| <b>Word Speed</b>   | This setting is used when "Highlight words as read" is enabled. It determines how fast the words are read.   |
| <b>Icon Size</b>  | Set your preferred Icon Size. Note, with the larger sizes, the bottom of the toolbar icons may be truncated.   |
| <b>Clipboard Speaker</b>  | Enable, or disable the clipboard speaker mode. If enabled, <b>Mobile Talker</b> will speak anything that is copied to the clipboard  |
| <b>Speak each word when typed</b>   | Enable, or disable the speak each word mode. If enabled, <b>Mobile Talker</b> will speak each word after it is typed. It only knows that you have typed a word if you enter a space, full stop or other punctuation character.   |
| <b>One handed operation</b>   | Enable, or disable the one handed operation mode. If enabled, <b>Mobile Talker</b> can be operated with one hand. In the Phrases tab, it will not speak the selected phrase until you press the play button or the devices enter button (this is usually the centre button of the arrow pad) |
| <b>Autostart SMS Catcher</b>  | Enable, or disable auto starting of SMS Catcher. If enabled, <b>Mobile Talker</b> will automatically start SMS Catcher when <b>Mobile Talker</b> starts.   |
| <b>Stop after 1 hour</b>  | Enable, or disable the auto stop mode. If enabled, <b>Mobile Talker</b> will automatically stop after running for one hour.  |
| <b>Read in Word Mode</b>  | Enable, or disable the read in word mode. If enabled, <b>Mobile Talker</b> will speak each word more slowly as a separate word, rather than as a complete sentence.  |
| <b>Speak from Cursor</b>  | Enable, or disable the speak from cursor mode. If enabled, <b>Mobile Talker</b> will start speaking words from the current cursor position. If a word or phrase is selected, it will speak this instead.   |



## Menus

**Context menus: Hold stylus/finger down on text fields, until the menu appears.**




|                                 |   |
|---------------------------------|---|
| <b>Undo</b>                     | Windows Undo feature (only applies to text fields)  |
| <b>Cut</b>                      | Windows Cut feature (only applies to text fields)   |
| <b>Copy</b>                     | Windows Copy feature (only applies to text fields)  |
| <b>Paste</b>                    | Windows Paste feature (only applies to text fields)   |
| <b>Delete</b>                   | Windows Delete feature (only applies to text fields)  |
| <b>Select All</b>               | Windows Select All feature (only applies to text fields)  |
| <b>New Category</b>             | <a href="#">New Category window</a> appears so that a new category can be added.  |
| <b>New Phrase</b>               | Displays the <a href="#">Phrases Window</a> . Select the category for the new Phrase, and enter the text. Then press the Save button. |
| <b>Delete Selected Category</b> | Deletes selected category   |
| <b>Delete Selected Phrase</b>   | Deletes selected phrase   |
| <b>Edit Phrase</b>              | Enables editing of the selected phrase. You can change the category or the text of the phrase.  |

## Other Windows






### Send SMS

|   |   |
|---|---|
| <b>Phone Number</b>   | Choose a category under which the phrase should be stored. To add new categories use the Edit menu. |
| <b>Contacts</b>   | Input the text you wish to store.   |
| <b>Keypad</b>   | Quit this window without making any changes.  |
| <br><b>Cancel button</b> | Cancel and return to the last tab.  |
| <br><b>Send button</b>   | Send the SMS message.   |






### New Category

|  |  |
|--|--|
| <b>Category Name Input Box</b>   | Enter a name for this category                   |
| <br><b>Cancel button</b>    | Cancel and return to Phrases tab.                |
| <br><b>Keyboard button</b> | Press to display or Hide the on-screen keyboard. |
| <br><b>Save button</b>    | Save the new category.                           |

### New or Edit Phases

|   |   |
|---|---|
| <b>Select Category</b>  | Choose a category under which the phrase should be stored. To add new categories use the button below.                    |
| <br><b>New Category</b>    | tap the button then input the Category name.  |
| <b>Select Icon List</b>   | Choose which Icon you would like to represent the phrase.   |
| <b>Phrase Input Box</b>   | Input or edit the phrase.   |
| <br><b>Cancel button</b>   | Cancel and return to Phrases tab.   |
| <br><b>Clear button</b>    | Press to clear the Phrase Input Box. If some of the text is selected, the clear button will only clear the selected text. |
| <br><b>Keyboard button</b> | Press to display or Hide the on-screen keyboard.  |
| <br><b>Save button</b>     | Save the new phrase.  |

## Pronunciation

|  |   |
|--|---|
| Select or Enter New Word   | The dropdown list will be populated with any existing pronunciation commands.   |
| <br>New button      | Press to input a new word for which you can then define the pronunciation.  |
| Pronunciation Command Text Input Box   | Enter the pronunciation command. Ask your supplier for documentation for the installed Text-to-speech engine. To test the pronunciation, press the  (Play) button. |
| <br>Cancel button   | Cancel and return to Phrases tab.   |
| <br>Keyboard button | Press to display or Hide the on-screen keyboard.  |
| <br>Save button     | Saves the new pronunciation for this word.  |

## Other Information

|  |  |
|--|--|
|  |  |
|--|--|